

## Appendix 5.1 - Instructions for use (Toric vision)

### **IMPORTANT: Please read carefully and keep this information for future use.**

Have your eyes examined regularly and follow your Eye Care Practitioner instruction for the proper use and care of your lenses. Please read the PATIENT INSTRUCTION GUIDE for important safety information before use. Ask your Licensed Eye Care Practitioner for a copy of PATIENT INSTRUCTION GUIDE or visit [www.migwang.com](http://www.migwang.com)

**Rx Only**


**CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a Licensed Eye Care Practitioner.**

### **PRIOR TO LENS INSERTION**

Wash and dry your hands before removing the lens from its container. Do not allow tap water to come into contact with the lens.

**ATTENTION: Personal hygiene is very important.**

#### **INSERTION:**

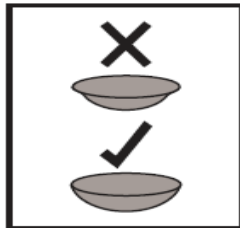

Ensure the lens is not inside out.




Draw down the lower eye lid and pull the upper eye lid upwards and place the lens gently over the visible corneal.

#### **LENS REMOVAL:**



Downward pressure on the lower eye lid whilst looking up will tend to move the lens downwards.

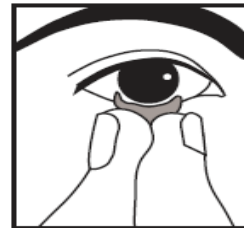

The lens can then be pinched off the eye using the thumb and forefinger.

### **DISINFECTION**

Before inserting the lens and after removing it from the eyes, always rinse the lens with the contact lens solution suggested by your Licensed Eye Care Practitioner. After wearing your lenses, store them in a case filled with fresh contact lens solution. If you wear contact lenses infrequently, change the contact lens solution in the case with fresh solution once a week.

### **WARNING AND PRECAUTIONS**

Remember regular check-up and always follow the advice of your Licensed Eye Care Practitioner. Do not lend your lenses to anyone; they are only for one patient for a time. Follow the wearing and replacement schedule as prescribed by your Licensed Eye Care Practitioner.

**IF YOU EXPERIENCE EYE DISCOMFORT, EXCESSIVE TEARING, VISION CHANGES, OR REDNESS OF THE EYE, IMMEDIATELY REMOVE YOUR LENS AND PROMPTLY CONTACT YOUR LICENSED EYE CARE PRACTITIONER.**

These instructions do not give you all the information you need for wearing contact lenses. It is very important to consult with your Licensed Eye Care Practitioner, and follow the prescription.

### **REPORTING OF ADVERSE REACTIONS**

All serious adverse experiences and adverse reactions observed in patients wearing this contact lens or experienced with the lens should be reported to:

Mi Gwang Contact Lens Co.,Ltd.  
693, Namcheon-ro, Namcheon-myeon, Gyeongsan-si,  
Gyeongsangbuk-do, Korea  
Email: [KRA@clearlab.com](mailto:KRA@clearlab.com)